

PerCept™/MC

Concentrated General Virucide Disinfectant Cleaner



An effective low-level disinfectant for non-critical medical devices and nonporous, hard environmental surfaces.

Effective

- ▶ Exceptional cleaning on stainless steel, porcelain, chrome and other common surfaces
- ▶ Ideal for use in hospitals, long-term care facilities, schools and daycares

Easy-to-Use

- ▶ Fragrance-free, Volatile Organic Compound (VOC) free, Dye-free
- ▶ Not manufactured using APE's/NPE's
- ▶ Non-corrosive and non-irritating to skin and eyes at use dilution as per OECD criteria

Designed For

- ▶ Healthcare
- ▶ Education
- ▶ Industrial
- ▶ BSC
- ▶ Retail
- ▶ Lodging
- ▶ Government



Use Overview

PerCept™/™ Concentrated General Virucide Disinfectant Cleaner is proven effective as a low level disinfectant for non-critical medical devices and nonporous, hard environmental surfaces in facilities such as daycares, schools, call centers, long-term care facilities, hospitals and emergency services. It can safely be used as a broad-spectrum disinfectant.

Disinfection of Non-Critical Medical Devices & Nonporous, Hard Surfaces

- ▶ Pre-clean heavily soiled areas.
- ▶ Dilute 1:16. Apply to surface with cloth or disposable wipe. Ensure surface remains wet for 5 minutes at 20°C. Wipe dry or rinse.
- ▶ Food contact surfaces require rinsing with potable water.

Cleaning & Disinfecting Surfaces of Blood and Body Fluids

- ▶ Wear gloves and remove excess blood and fluid with absorbent materials.
- ▶ To clean contaminated area, apply a 1:16 solution to the surface. Soak for 30 seconds and wipe dry.
- ▶ To disinfect contaminated area, apply a 1:16 solution to the surface. Ensure surface remains wet for five minutes. Wipe surface dry or rinse.



Broad-Spectrum Sanitizing

- ▶ Dilute to 1:64 or 1:128 and apply to surface, allow surface to remain wet for 30 seconds. Wipe surface dry or rinse.
- ▶ Food contact surfaces require rinsing with potable water.

Cleaning & Sanitizing Critical Care Floors

- ▶ Dilute 1:64 or 1:128. Use mop or autoscrubber and allow to air dry. No rinsing required.

Product Specifications

Description	
Color/Form	Colorless liquid
DIN	02241255
Flash Point	> 200° F (> 93.4° C)
pH	0.7 (Concentrate) 1.8 (Use Dilution 1:16)
Scent	Characteristic
Use Dilution	1:16 (Disinfection) 1:64 (Sanitizer) 1:128 (Sanitizer)
VOC	0%
Shelf Life	1 year
Solubility	Complete
Specific Gravity	1.069

Available items

Product Code	Description/Package Size	Dilution
4337151	2 x 1.5 gallon/ 5.67 L Command Center™/™	1:16
4337041	4 x 4 L / 1.05 gallon Container	1:16, 1:64, 1:128

Hazard Rating

HMIS	Concentrate	NFPA	HMIS	Use Dilution (1:16)	NFPA
3	Health	3	0	Health	0
0	Flammability	0	0	Flammability	0
0	Reactivity	0	0	Reactivity	0

4=Very High; 3=High; 2=Moderate; 1=Slight; 0=Insignificant.

Safety Reminder

Please make sure your employees read and understand the product label and Material Safety Data Sheet before using this product. The label contains directions for use, and both the label and MSDS contain hazard warnings, precautionary statements and first aid procedures. MSDS are available on-line at www.diversey.com or by calling 888 352 2249.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.

For additional information, please contact us at +1 800 558 2332 or at www.diversey.com

© 2011 Diversey, Inc., 8310 16th Street, Sturtevant, WI 53177-1964. All Rights Reserved. SPC10106 (11/325)

Diversey
for a cleaner, healthier future™