



Rescue™/MC Wipes Sporocidal

Ready-to-Use sporicide for hard, non-porous surfaces that is effective against *Clostridium difficile*, *Clostridium sporogenes* and *Bacillus subtilis*, utilizing AHP® technology.

Features & Benefits

- Ready-to-Use sporicide powered by AHP® technology
- Effective against *Clostridium difficile*, *Clostridium sporogenes*, *Bacillus subtilis* in 10 minutes, providing cleaning and disinfection on high touch surfaces
- The AHP® active breaks down to oxygen and water after use

Applications

- Designed for cleaning and disinfection of high touch surfaces, and the larger wipe size allows for large environmental surface coverage





Rescue^{TM/MC} Wipes Sporidical

Use Directions*

Heavily Soiled Surfaces Require Cleaning Prior to Disinfection

Personal Protection: Disposable gloves, gowns, face masks, or eye coverings as appropriate, must be worn during all cleaning of body fluids, and decontamination procedures.

Cleaning Surfaces Contaminated with Body Fluids: Appropriate Personal Protection must be worn during all cleaning and disinfection procedures. Remove excess body fluids with absorbent materials. Apply to surfaces using the disposable wipe, let stand for 1 minute. Disinfect contaminated area.

Sporidical Disinfection of Hard Non-porous Surfaces: This product is NOT to be used as a terminal sterilant/High Level Disinfectant for critical instruments and medical devices. This product is intended for use as an Environmental Surface Sporicide on hard non-porous surfaces of *C. difficile* patient rooms.

- Apply to surfaces using the disposable wipe.
- To achieve Sporidical disinfection ensure surface remains wet for 10 minutes.
- Rinse after disinfection to remove all residue.

Note: This product is not intended for use on medical equipment without device manufacturer approval.

Disposal of Infectious Material: Products contaminated with blood or body fluids should be disposed of according to Federal, Provincial, and local regulations for infectious waste disposal.

* Refer to product label for complete use instructions.

Technical data	Rescue ^{TM/MC} Wipes Sporidical
DIN	02447614
Color/Form	Colorless Wipes
pH	2.7
Scent	Characteristic of ingredients
Shelf Life	2 Years

Product	Pack size	Dilution	Product code	
Rescue ^{TM/MC} Wipes Sporidical	12 x 160 count / 20.32 cm x 22.86 cm Wipes / 8 in. x 9 in. Canisters	Ready-to-use	100906725	

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

AHP® and Design are registered trademarks of Virox Technologies, Inc. used under license.

www.diversey.com

© 2018 Diversey, Inc. All Rights Reserved. PSS90515 (18/192)