



Virox™/MC 5

Concentrated Surface Cleaner & Disinfectant

A one-step disinfectant cleaner based on proprietary Accelerated Hydrogen Peroxide technology® to deliver fast and effective cleaning performance.

Features & Benefits

- Disinfects in 5 minutes at 1:16 dilution
- Provides broad spectrum sanitizing on hard surfaces in 30 seconds
- Proven effectiveness against the Polio virus, Sabin strain type 1 (ATCC VR 192) allows for a General Virucide claim against most enveloped and non-enveloped viruses
- Kills Norovirus, MRSA and VRE in 5 minutes
- Environmentally-responsible; no VOC's, no added fragrance, no NPE's
- The active ingredient breaks down to water and oxygen after use

Applications

- For use as cleaner and disinfectant of hard, non-porous surfaces



**Virox™/MC 5****Concentrated Surface Cleaner & Disinfectant****Use instructions****Disinfectant Use Directions:**

- Dilute concentrate at 1:16.
- Disinfection of non-critical items, equipment and hard surfaces coming in contact with intact skin such as the exterior of hemodialysis machines, stethoscopes, tabletops, etc.
- Apply to surface with cloth or disposable wipe. Ensure surface remains wet for 5 minutes at 20° C.
- Cleaning and Disinfecting Surfaces of Blood and Body Fluids that May Contain HIV: Gloves should be worn. Remove excess blood and fluid with absorbent materials. Clean Contaminated Area: Apply diluted 1:16 solution to surface, soak 30 seconds and wipe dry. Disinfect Contaminated Area: Apply diluted 1:16 solution to surface, allow to remain wet for 5 minutes. Wipe surface dry or rinse.

Broad-Spectrum Sanitizing (99.999%) on Environmental Surfaces

- Dilute concentrate at 1:64 or 1:128
- Apply to surface and allow to remain wet for 30 seconds. Wipe dry. No rinsing is required. (Food contact surfaces require rinsing with potable water.)

Technical data	Virox™/MC 5
DIN	02239828
Color/Form	Colorless, liquid
pH	0.5–1.0 (Concentrate) 1.5–2.0 (Use Dilution)
Scent	Characteristic
Shelf Life	2 years (Concentrate) 28 days (Use Dilution)

Product	Pack size	Dilution	Product code	
Virox™/MC 5	2 x 2.5 L / 84.5 oz. J-Fill® Bottles	1:16	2963741	🇨🇦
Virox™/MC 5	4 x 3.78 L / 1 U.S. Gal. Containers	1:16, 1:64, 1:128	101103735	🇨🇦

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.